


Assembly Instructions

A: Place the strap through the slot marked 1st from the side where the forehead supports are proud.

Place the strap back through the slot marked 2nd.

B: Place the peak of the hardhat between the two rows of visor supports and then pull the strap over the hard and adjust to suit. Do not overtighten.

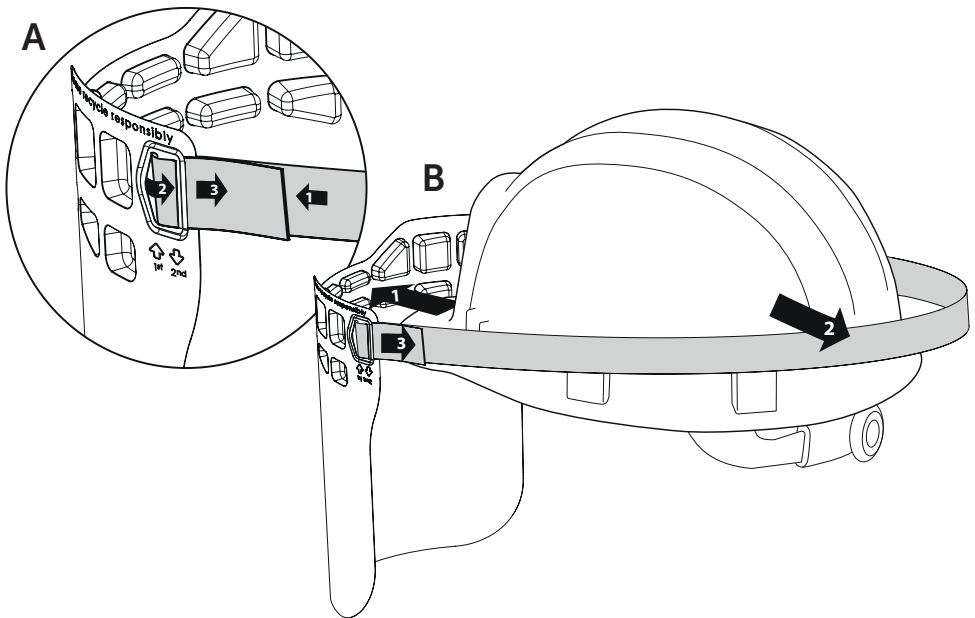

WARNING: This visor offers no impact resistance and is purely intended for hygiene purposes and must not be used in place of full protective visors.

Hygiene instructions

Visor can be rinsed with hot soapy water using a soft cloth or follow the recommended guideline of your place of work.

Strap contains latex.

Class I non-sterile device Expiry date : End 2021

To reorder visit macpac.co.uk/facevisor or your local stockist.