

This protective faceshield is manufactured for COVID-protection only.

This protective faceshield is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19

Assembly Instructions

Place the visor on a flat surface with the engraved text in the correct reading direction facing downwards

1. Squeeze the sides of the foam strip at either end firmly enough to create the backing paper. Then peel up the paper from the crease to remove.

2. Place the foam with the adhesive side facing down over the hatched area and press firmly to secure.

3. Place each side of the strap through the slots marked 1st from the foam side and then thread back through the slot marked 2nd.

When placed on your forehead pull the straps to tighten to give the most comfortable fit.

Hygiene instructions

Visor can be wiped down with a soft cloth and hot soapy water or follow the recommended guidelines from your place of work.

